
CardioMessenger® Smart

de	Gebrauchsanweisung
en	Technical Manual
es	Manual técnico
fr	Manuel technique
it	Manuale tecnico di istruzioni
nl	Technische handleiding

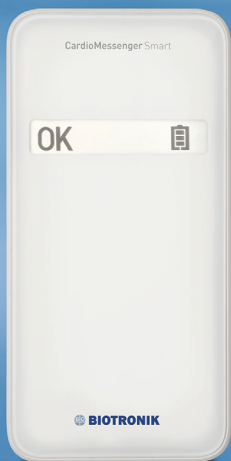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1 Introduction

Dear Patient:

You have received a device with the additional Home Monitoring function by BIOTRONIK.


How Home Monitoring works

Your device (1) is equipped with a special transmitter which sends cardiac information to your Cardio-Messenger (2). This usually happens at night.

The transmission power from your device is low and does not impair your health in any way. Its limited transmission range, however, requires the use of the Cardio-Messenger.

The CardioMessenger collects the information and transmits it to the BIOTRONIK Service Center (4) as encoded messages via a mobile connection (3).

Here, the messages are decoded and can be viewed by your physician (5) on a protected web site.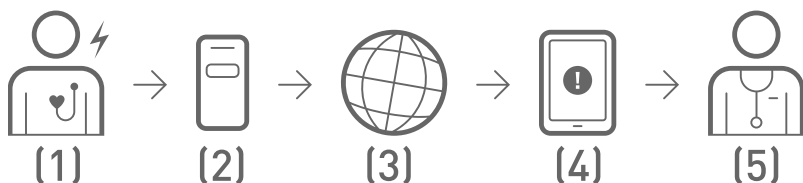

Usage of this additional data is specific to the patient and the implanted device. Your physician will explain to you how he or she will use the Home Monitoring function.

Note: Home Monitoring is not an emergency system. If you are not feeling well, contact a physician.

2 First Steps

Check the Package Contents

Your CardioMessenger is supplied ready for use, and you can operate it immediately by inserting the power plug into the wall outlet.


However, prior to usage, check the CardioMessenger and its accessories for any visible damage and use only undamaged components.

Return a damaged CardioMessenger to your physician.


Use only the provided original power supply brick (see the technical data).

Other equipment may impair proper functioning of the CardioMessenger and increase the emitted interference or decrease the CardioMessenger's resistance to interference.

The product package includes the CardioMessenger (1) with its technical manual (2), the power supply brick with power cord and power plug (3), and the quick reference guide (4).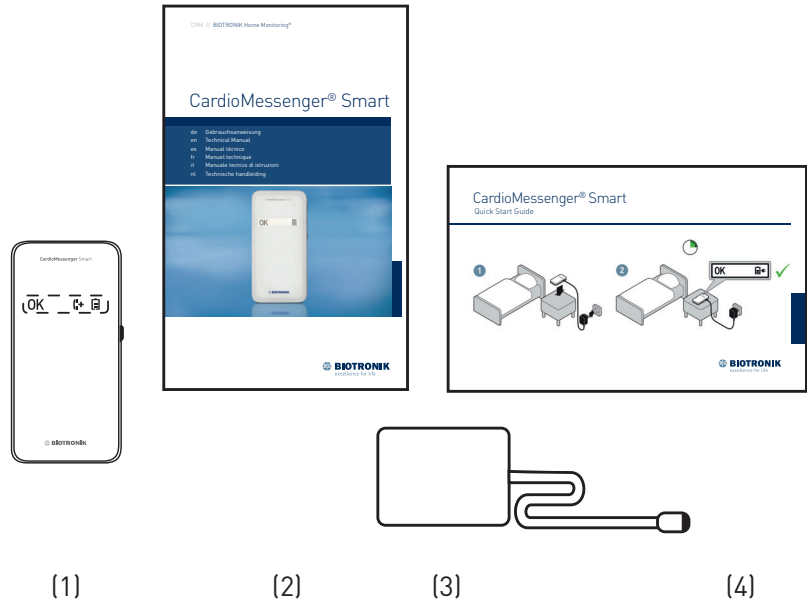

Where Do I Put the CardioMessenger?

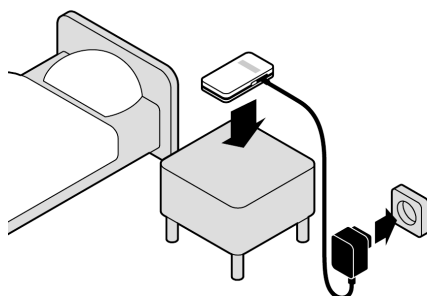At night, the CardioMessenger should be placed close to your bed to ensure the nightly data transfer from the implanted device.

The bedside table is therefore the best location for your CardioMessenger, as it usually meets the following conditions:

- The CardioMessenger is placed on a solid base and cannot fall.
- The distance to the implanted device is less than 2 m (6 feet), so that regular data transmission from the implanted device to the CardioMessenger at night is assured.
- Positioned on the bedside table, you can clearly see the symbols on the CardioMessenger's display. This enables you to verify on a daily basis that it is ready for service.

However, **if the bedside table is made of metal**, you should not place the CardioMessenger directly on the table. For example, place a stack of books to establish a distance of approx. 5 cm between the two so that the metal does not interfere with the device's data reception.

Please take the following also into consideration:

- Do not place the CardioMessenger next to a television set, microwave oven, or a similar source of electro-magnetic interference.

You may hear noises typical of cellular phones if you place the CardioMessenger too close to a radio alarm or a television set, for example.


- Do not expose the CardioMessenger to temperatures exceeding 40°C (104°F). Do not put it in places where it is subjected to direct sunlight and do not put it directly under a halogen spotlight.
- Do not expose the CardioMessenger to temperatures below negative 5°C (23°F). The power supply brick should not be exposed to temperatures lower than 0°C (32°F).
- Protect the CardioMessenger against water and high humidity.
Do not place it in the bathroom.

How Do I Connect the CardioMessenger?

Your CardioMessenger is already configured and is ready for use. You can operate it immediately by inserting the power plug (see Appendix, p. 95 for the type) into the wall outlet.

Note: Also refer to the included quick reference guide, particularly consult the depiction on how to change the UK adapter.


However, proceed as follows if the plug has been disconnected during removal from the package or during shipping:

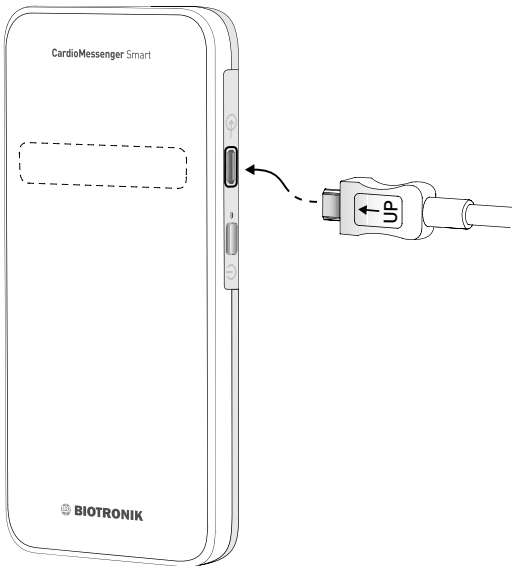1	Connect the small plug (micro USB plug) on the right to the CardioMessenger. Make sure that the marking (white arrow) is on the upper side of the plug.  The connector port is labeled with the following symbol: 
2	Insert the power plug into the wall outlet.

Make sure that the outlet is easily accessible and not connected to a light switch in order to prevent the CardioMessenger from accidentally being turned off.

The CardioMessenger now turns on automatically and performs a self-test.

The CardioMessenger is ready for use once the self-test is completed and the following icons are displayed:

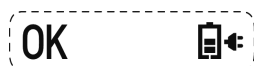If this is not the case, please refer to: Error Resolution, p. 81.

How Do I Use the CardioMessenger?

The CardioMessenger automatically receives the information from your implanted device and transmits it to the BIOTRONIK Service Center. Therefore, there are only a few points that need to be considered.

Check once a day whether your CardioMessenger is switched on and ready for use.

This is indicated by the following icons:

If you want to use the CardioMessenger in mobile operation, we recommend that you make a habit of charging it every night on the bedside table.

How Do I Turn Off the CardioMessenger?

Since the CardioMessenger contains a mobile (“cellular”) module, you may need to switch off the CardioMessenger for safety reasons in areas where the use of mobile phones is prohibited (e.g., in an airplane).


Such areas can be identified by the following or similar signs:

In some locations, the use of cellular phones is prohibited to provide quiet zones (e.g., in a theater or cinema). As the CardioMessenger is silent, it does not need to be turned off in such locations.

Press and hold the blue key on the right of the CardioMessenger for two seconds until the display turns off.

The blue key is labeled with the following symbol: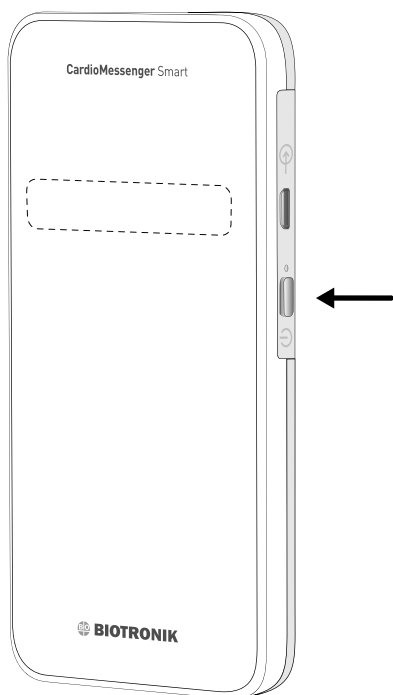

The CardioMessenger is switched off.

Note: The functions of your implanted device are not affected by the CardioMessenger at any time. Your implanted device remains fully functional even if the CardioMessenger is not ready for use.

3 The CardioMessenger Icons

The CardioMessenger has the following icons:

1		Operation icon
2		Call back icon See Call Back Function, p. 79
3		Information icon See Error Resolution, p. 81
4		The battery icon is always displayed with 1-3 bars according to the charging status.
5		When the CardioMessenger is connected to the power supply brick and charging, a battery icon with a small power plug is displayed.

4 Functions

Self-Test

The CardioMessenger automatically conducts a self-test after being connected.

1	All icons on the CardioMessenger are displayed.  The CardioMessenger then checks the connection to the cellular phone network.
2	The operation icon flashes and the battery icon is displayed.  The connection test can take up to 15 minutes.
3	Once the connection is established, the operation and battery icons remain permanently activated.  The CardioMessenger is now ready for use.

If the connection was **not established**, the information icon flashes.

Additional information can be found under: Error Resolution, p. 81.

Call Back Function

The call back function is an additional function that your physician can use in different ways. You will be informed by your physician if and how he or she will use this function.

For example, your physician can use the call back button to ask you for a call. Your physician can turn on the icon via the cellular phone network. It will then flash for a maximum of three days.

Contact your physician during office hours as soon as you notice that the call back icon is flashing.

Turning off the call back icon

To turn off the call back icon, briefly turn off the CardioMessenger.

1	Press the blue key on the right side of the CardioMessenger for about two seconds.
2	All symbols disappear.
3	Wait approximately ten seconds.
4	Press the blue key again for approximately two seconds.

Note: If the CardioMessenger is connected to the power supply brick, it starts automatically. You neither have to wait nor switch it on.

The CardioMessenger performs a self-test.

The operation and battery icon is then displayed, and the call back icon stops flashing.

However, please do not forget to call your physician.

5 Error Resolution

If your physician contacts you because device messages are not being received but your CardioMessenger was ready for use during the period in question, you should remove possible sources of radio interference from the immediate vicinity of the CardioMessenger.

Possible sources of interference can be communication devices such as wireless home network equipment, cellular phones, cordless phones, and their base stations.

In the IEC 60601-1-2: 2007 standard, a distance of 3.3 m (3.6 yd) is recommended for these cases. (For more information, refer to Appendix, p. 95).

Malfunctions on the CardioMessenger are indicated by the icons.

Icon	Behavior	Operational status
	Off	Error type A: No power supply
	Flashing	Error type B: Self-test failed
	Flashing	Error type C: No mobile connection

Error type A

No power supply

The operation icon is not displayed. The Cardio-Messenger is not ready for use.

- Make sure the micro USB plug is properly inserted into the CardioMessenger.
- Make sure the power plug is properly inserted into the wall outlet.
- Make sure the outlet provides an electrical current, for example by temporarily connecting the bedside lamp to the outlet and turning the lamp on.

If you do not find any errors, contact your physician.

Error type B

Self-test failed

All symbols are flashing.

Repeat the self-test as the CardioMessenger did not finish it.

The CardioMessenger is not ready for use.

1	Disconnect the CardioMessenger from the power supply brick if relevant.
2	Press the blue key on the right side of the CardioMessenger for about two seconds.
3	Leave the CardioMessenger turned off for about thirty seconds.
4	Turn on the CardioMessenger by connecting it to the power supply.

The CardioMessenger starts and automatically repeats the self-test. When the self-test is completed, the CardioMessenger is ready for use.


Note: The connection test can take up to 15 minutes.

If all symbols continue to flash, the CardioMessenger is defective. Return it to your physician.


Error type C

No mobile connection

The information icon flashes and the battery icon is displayed. Check the mobile connection since the Cardio-Messenger cannot connect to the BIOTRONIK Service Center.

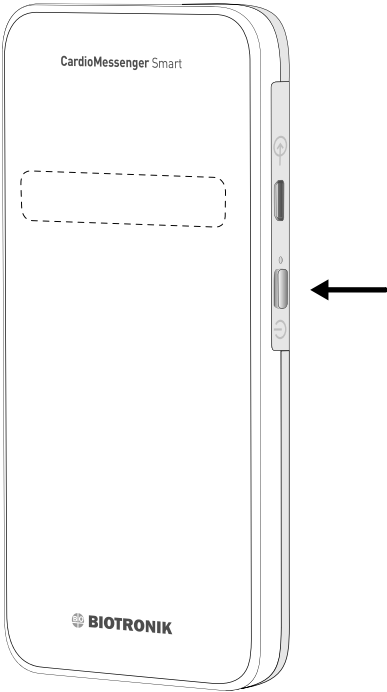1	Press the blue key on the right side of the CardioMessenger for about two seconds. 
2	Put the CardioMessenger in a place with better mobile reception. Make sure that the distance to the device is still less than 2 m (6 feet).
3	Press the blue key again for approximately two seconds.

The CardioMessenger restarts and performs the self-test. It checks the connection to the cellular phone network. The operation and battery icon is displayed once the connection test is completed successfully. The CardioMessenger is now ready for use.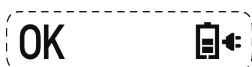

The connection test can take up to 15 minutes.

If the CardioMessenger is generally unable to connect to the cellular phone network from near your bed, contact your physician.

Note: Missing cellular phone network connection can occur in rooms with thick walls or when travelling.

6 Handling

The CardioMessenger is intended primarily for continuous operation at home because it receives information from your implanted device once daily, usually at night, and forwards it to the BIOTRONIK Service Center.

Make a habit of charging the CardioMessenger at night so that it can be used in mobile operation during the day.

Fully charged, you can use the CardioMessenger while mobile for up to 48 hours, but note the following:

- Protect the CardioMessenger from direct contact with water. For example, wear it under your coat or keep it in a bag when it rains.
- Do not carry the CardioMessenger inside the breast pocket of your shirt or jacket as the distance from here to the implanted device could be less than 15 cm.
- To prevent skin irritations, do not carry the CardioMessenger directly on your skin.
- Do not bring the CardioMessenger into the vicinity of fire.
- Do not expose the CardioMessenger to temperatures below negative 5°C (23°F).
Battery longevity for the CardioMessenger may decrease at lower temperatures.
- Do not turn on the CardioMessenger if it has recently been in a cold environment. Let it warm up slowly to room temperature, since the resulting condensed water may harm the electronic circuitry.

- Do not operate the CardioMessenger in areas where cellular phones are **prohibited for safety reasons** (for example, in certain areas of the hospital or in airplanes).

** WARNING**

The distance between the CardioMessenger and the device must be at least 15 cm (6 inches) so that the CardioMessenger does not interfere with the device.

The CardioMessenger contains a mobile (“cellular”) module. In order to prevent any interference with your implanted device, the prescribed minimum distance between the device and a cellular phone must also be maintained between the device and the Cardio-Messenger.

To disconnect the CardioMessenger from the alternating current supply, pull the power supply brick plug out of the socket.

Charging

You should charge the CardioMessenger once before using it for the first time.

It is also recommended to operate the CardioMessenger with the power supply brick on the bedside table to fully charge the battery.

Note: Do not charge the CardioMessenger with the power supply brick in the outdoors.


At the very latest, the CardioMessenger must be charged when the battery icon flashes.

Proceed as follows:

1	Connect the CardioMessenger to the mains supply.
---	--

During charging, the individual segments of the battery icon flash alternately and a small power plug is displayed.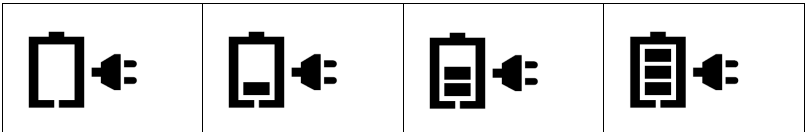

The three bars on the battery icon flash successively until the CardioMessenger is fully charged. Once it is fully charged, all three bars are completely filled. The charging process usually takes three hours.

Note: The CardioMessenger can be used with the power supply brick if the battery is defective.
Even if the battery is completely discharged, the Cardio-Messenger can still operate using the power supply brick.

If handled properly, the installed battery should supply the CardioMessenger with 48 hours of power even after 500 complete charging cycles (which is at least two years).

Cleaning

- Keep the CardioMessenger clean and away from dirty or dusty environments.
- Use a soft, lint-free cloth for cleaning.

⚠ WARNING
Turn off the CardioMessenger before cleaning it with a damp cloth.

- Use a cloth slightly moistened with water for cleaning. However, avoid bringing the CardioMessenger into direct contact with water or solvents.
- Protect the CardioMessenger from direct contact with water.

Maintenance

The CardioMessenger is intended for continuous, automatic operation. Once correctly installed, ongoing maintenance typically is not required.

Disposal

Do not dispose of the CardioMessenger with your household trash.

CardioMessenger and the associated power supply brick contain materials that must be correctly disposed of in accordance with environmental protection regulations.

You may dispose of the CardioMessenger and its associated power supply brick as electronic waste in accordance with the applicable regulations if you no longer use it or if it is defective.

The CardioMessenger and all the parts from the package can be returned to your physician. Your physician will return all parts to BIOTRONIK.

BIOTRONIK ensures disposal in accordance with the national versions of the European guideline 2012/19/EC on waste electrical and electronic equipment (WEEE 2).

7 Precautionary Measures

The CardioMessenger is a medical product and therefore complies with the strict requirements for the development, manufacturing, and testing of medical devices.

Please take the following precautions:

Statutory regulations for electrical devices in hospitals require that the CardioMessenger and its accessories not be used in areas defined as patient environment (e.g., in the operating room).

Protect the CardioMessenger and the power supply brick from:

- Water and high humidity
- Temperatures above 40°C (104°F) (e.g., direct sunlight, strong halogen spotlights, fire)
- Temperatures below negative 5°C (23°F; CardioMessenger) and below 0°C (32°F; power supply brick)
- Solvents, acids, detergents, and lyes
- Pressure below 700 hPa (corresponding to altitudes above 3000 m, approx. 10,000 feet)
- Pressure above 1060 hPa (corresponding to altitudes below sea level)
- Violent shocks or other strong mechanical influences
- Intense light sources (direct sunlight, strong halogen spotlights)

WARNING

Lay the power cord so that there is no risk of strangulation.

WARNING

Keep the CardioMessenger away from children under three years of age due to ingestible small parts.

8 Guidelines


Europe

Your device transmits diagnostic data to the CardioMessenger via a radio frequency (RF) assigned by the European Conference of Postal and Telecommunications Administration for the operation of Ultra Low Power Active Medical Implants (CEPT/ERC REC 70-03).

BIOTRONIK is legally obligated to inform you that the radio service does not have exclusive use of the assigned frequencies and that the transmission of device data is not permitted to interfere with other radio services. The frequency and technical parameters of the built-in transmitter have been carefully selected to ensure that interference between other services and the data transmission of the device is unlikely.

Furthermore, BIOTRONIK is obligated to inform you that the regulatory agency can withdraw the frequency allocation and prohibit the radio service between the device and CardioMessenger. Since this service is currently established throughout Europe and North America, withdrawal of the frequency allocation is not expected in the foreseeable future.

The CardioMessenger, like the implantable device itself, has been evaluated by an independent testing authority for its compliance with statutory regulations. The CardioMessenger carries the following approval mark:

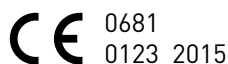In addition, the CardioMessenger contains a radio modem that connects to the cellular network at the frequencies of 850/900/1800/1900/1900 MHz. BIOTRONIK uses the radio modem in accordance with the manufacturer's specifications and in compliance with the approval requirements.

The radio modem has been evaluated and approved by an independent authority for its compliance with the statutory regulations. As an indication of this, it carries the following approval mark

Electromagnetic compatibility

Note: The CardioMessenger is protected from disturbances resulting from electromagnetic interference, electrostatic discharges, and other sources of interference – including interference induced by wiring. At the same time, interfering electromagnetic emissions from the CardioMessenger have been minimized. The CardioMessenger therefore meets the requirements of EN 60601-1-2 in every respect.

Note: Other equipment, for example portable and mobile RF radiocommunications equipment, may also interfere with the CardioMessenger, even if this equipment complies with CISPR emission requirements. However, this possible interference does not affect the functionality of the device.

Warranty

The CardioMessenger and all original components by BIOTRONIK are not subject to warranty when modified, used other than intended, stored improperly, or transported incorrectly. Only use the original packaging for shipment.

9 Appendix

Technical Data

General

- Operating mode: Continuous operation
- Longevity: 6 years
- IP 22

Permissible ambient conditions

- Operating temperature: -5°C to +40°C (23°F to 104°F)
- Storage and transport temperature: -20°C to +60°C (-4°F to 140°F)
- Store in a dry place:
Relative humidity: 30% to 75% (non-condensing)
- Atmospheric pressure: 700 hPa to 1060 hPa

CardioMessenger

- Type: CardioMessenger Smart
- Dimensions (L x W x H): approx. 130 x 65 x 17 mm (5.1 x 2.6 x 0.7 inches)
- Weight: approx. 127 g
- MICS frequencies: 402–405 MHz, FSK modulation
- MICS transmission power: 25 µW EIRP
- GSM frequencies: 850 MHz, 900 MHz, 1800 MHz, 1900 MHz
- GSM transmission power: 2 watts (850/900 MHz); 1 watt (1800/1900 MHz)

Power supply brick

- Type: FRIWO FW7520/05
- Dimensions (WxHxD): approx. 51.5 x 87.5 x 34 mm (2.0 x 3.4 x 1.3 inches)
- Housing type: MMP 15
- Housing material: PC / ABS VO 125
- Input voltage: 100–240 V AC at 50–60 Hz
- Output voltage: 5 V DC; 3 A
- Power cord length: approx. 1.83 m (2 yards)
- Power cord type: Micro USB-B
- Operating temperature: 0°C to +40°C (32°F to 104°F)
- Storage and transport temperature: -40 °C to +70 °C (-40°F to 158°F)
- Store in a dry place:
Relative humidity: 30% to 75% (non-condensing)
- Atmospheric pressure: 700 hPa to 1060 hPa

Battery (integrated)

- Type: Lithium-ion

Symbols on the Device

The label icons on the CardioMessenger symbolize the following:

	Observe the technical manual
	Store in a dry place
	On and off key (standby)
	Port for micro USB connector

Legend for the Label

The label icons symbolize the following:

	BIOTRONIK order number
	Serial number
	Manufacturing date
	Follow the instructions for use!
	Storage temperature

	Air pressure limit
	Humidity limit
TP2	Compabiltiy with telemetry protocol version 2 of BIOTRONIK Home Monitoring
	Transceiver frequency
	This device contains material that requires special waste disposal according to the environmental protection guidelines. The European Directive 2012/19/EC on waste electrical and electronic equipment (WEEE 2) must be observed. Return devices that are no longer used to BIOTRONIK.
	CE mark
	Device
	Contents
	CardioMessenger Smart
	Power supply brick

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All rights reserved.
® Registered trademarks of BIOTRONIK SE & Co. KG: